

# 4Seal<sup>®</sup>

## Hemostatic Powder

**Rapid hemostasis at hand**



PROVIDING SPECIALISTS WITH  
**INNOVATIVE**  
HEALTHCARE SOLUTIONS

WITH THE CENTRAL FOCUS ON  
**PATIENTS' SAFETY** AND USERS' COMFORT



## 4SEAL<sup>®</sup> Hemostatic Powder


4SEAL<sup>®</sup> Hemostatic Powder is ready to use, a plant-based, ultra-hydrophilic polysaccharide that effectively and quickly stops bleeding. Furthermore, 4SEAL<sup>®</sup> Hemostatic Powder also may be used for postoperative adhesion prevention.

4SEAL<sup>®</sup> Hemostatic Powder absorbs rapidly water from the blood. The dehydration process increases the concentration of red blood cells, platelets, and coagulation proteins at the bleeding site and thus accelerates the natural blood clotting process.

4SEAL<sup>®</sup> Hemostatic Powder is available in 1g, 3g, and 5g in comprehensive bellow bottles with an applicator, which are delivered to the sterile filed in a double barrier. Extended and laparoscopic applicators (from an external manufacturer) 14 and 38 cm long, respectively, matching the integrated applicator of the bellow bottle are also available in Grena offer. 4SEAL<sup>®</sup> Hemostatic Powder contains no animal nor human components and is resorbed in the body within a few days.

### KEY FEATURES:

- Biocompatible, ultra-hydrophilic, pure plant-based technology
- Extremely quick and effective in bleeding control
- Easy to use and precise application
- Easy to store
- Ready to use instantly
- Long shelf life
- Hemostasis in 2-3 min

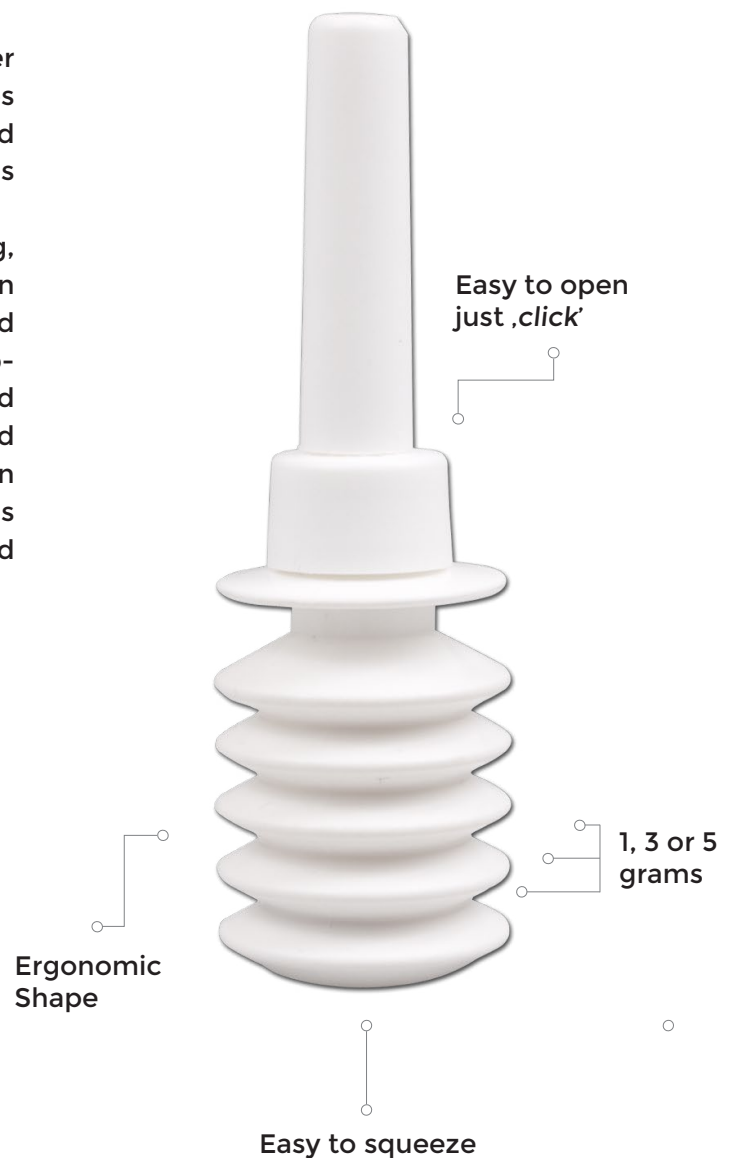







## **CLINICAL APPLICATIONS:**

4SEAL<sup>®</sup> Hemostatic Powder is intended for use in surgical procedures or injuries as an adjunct hemostat when control of bleeding from capillary venous, or arteriolar vessel by pressure, ligature and other conventional means is either ineffective or impractical. 4SEAL<sup>®</sup> Hemostatic Powder can be successfully used in many surgical specialties:


- **GENERAL SURGERY**
- **THORACIC SURGERY**
- **CARDIOVASCULAR SURGERY**
- **COLON AND RECTAL SURGERY**
- **GYNECOLOGY**
- **NEUROSURGERY**
- **ORAL AND MAXILLOFACIAL SURGERY**
- **ORTHOPEDIC SURGERY**
- **OTOLARYNGOLOGY**
- **PLASTIC AND MAXILLOFACIAL SURGERY**
- **UROLOGY**

REF	PRODUCT	DIMENSIONS	PCS PER BOX
1203-HP001	4SEAL <sup>®</sup> Hemostatic powder	4SEAL <sup>®</sup> Hemostatic powder 1 g	5
1203-HP003	4SEAL <sup>®</sup> Hemostatic powder	4SEAL <sup>®</sup> Hemostatic powder 3 g	5
1203-HP005	4SEAL <sup>®</sup> Hemostatic powder	4SEAL <sup>®</sup> Hemostatic powder 5 g	5

REF	PRODUCT	LENGTH (CM)		PCS PER BOX
1206-40700003	Extended Applicator	14		15
1206-40700002	Laparoscopic Applicator	38		15

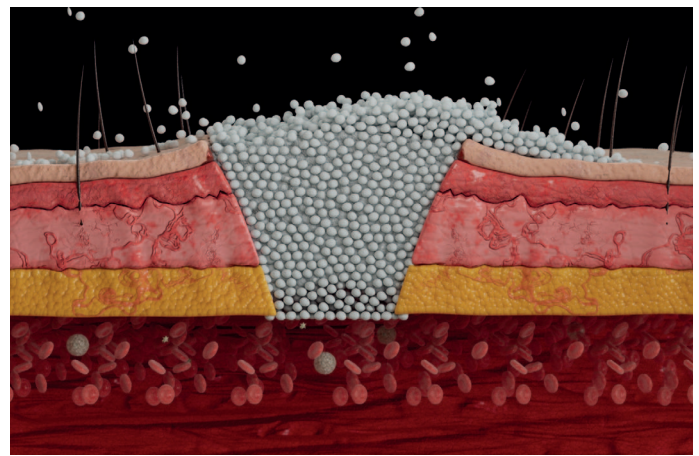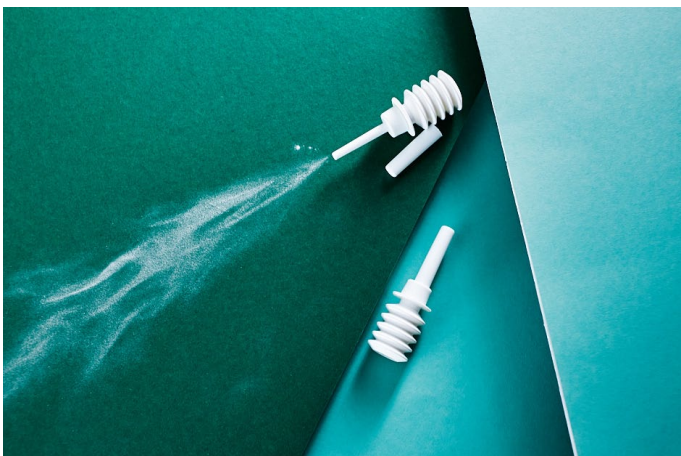

 **Grena Biomed Ltd.**  
Chelsea House, Chelsea Street,  
Nottingham, NG7 7HP,  
United Kingdom

 **MDML INTL LTD.**  
10 McCurtain Hill, Clonakilty, Co.  
Cork, P85 K230, Republic of Ireland

**CUSTOMER SERVICE CENTER**  
**NOTTINGHAM OFFICE, UK**  
☎ + 44 115 9704 800  
☎ + 44 115 9704 800  
✉ [export@grena.co.uk](mailto:export@grena.co.uk)  
**[www.grena.co.uk](http://www.grena.co.uk)**

